

Cleaning Instructions *pIRx3*

For cleaning your *pIRx3* we recommend the MK75 Destix, Disinfection- and Cleaning wipes.

They can be purchased in Switzerland from:

Global Tool Trading AG
Sonnenbühlweg 1
CH-6010 Kriens
Tel.: +41 (41) 342 02 77
Fax: +41 (41) 342 02 78
<http://www.gttag.ch>
info@gttag.ch

And in Germany from:

Volker Kunz
Kleinmann GmbH
Member of ITW Group
Am Trieb 13
D-72820 Sonnenbuehl
Tel.: +49 (0) 7128 9292-0
info@kleinmann.net



Clean your *pIRx3* with a clean cloth moistened with alcohol or isopropanol or with Destix MK75 disinfection wipes, according to the instructions. Never spray alcohol or other liquid onto the *pIRx3*, this could cause the material to swell. Rub the surface to be disinfected carefully but thoroughly and dry it after 1 minute contact time with a dry absorbent cloth.

The sensors must be kept clean because particles on the surface of the sensor windows can influence the temperature readings. Use a soft clean cloth slightly moistened with isopropanol or alcohol or a cotton swab to clean the sensors carefully.

The headband may be washed at 40°C with soap or detergent.